



INSTRUCTION FOR USE

GDT TEMP SOFT-RELINER

Self-curing Resin for Denture Reliner

INDICATIONS

As soft temporary denture reliner material and tissue conditioner.

INSTRUCTIONS FOR USE

1. Clean denture thoroughly by brushing with liquid detergent and water.
2. Inspect denture for proper occlusion.
3. Relieve all undercut and obvious pressure areas.
4. Add 5g of powder into the mixing cup provided. Using syringe, place 3ml of liquid (one full syringe, depending on the denture size the dosage may be doubled). Mix gently with a stainless steel spatula for 30 seconds.
5. After allowing the mixture to form a viscous, yet flowable state, pour the mixture into the denture, using a spatula to flow the material to the peripheral border to be certain that there is ample material to form an adequate peripheral (muscle-trimming) roll.
6. Rinse the mouth thoroughly with plain cold water.
7. Place the denture in the mouth and close the jaw slowly to a normal biting position to bring the teeth into proper occlusion. Hold this position for 1-2 minutes before muscle trimming.
8. After a further 3 minutes, remove the denture from the mouth and check for full coverage. Additional material can be added as required, as the material will blend homogeneously. If additional material is used, reinsert the denture for 2-3 minutes and re-inspect.
9. Trim excess with warm water and a sharp knife.
10. Visit dentist periodically for examination of progress during the resorption period. Additional material can be added as required after grinding and relieving areas where the base material has shown through.
11. Tissue Conditioner can be readily removed from a denture by scraping with a blunt knife or spatula and then wiping the denture with alcohol before washing the denture with a mild detergent and warm water.



CLEANING

Before sending dentures to a dental lab, always clean and disinfect following the instructions on the packet.

WARNING

Powder and liquid may cause slight irritation to eyes. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. The powder contains polymerised methacrylate and therefore may cause sensitisation by contact in susceptible persons. If sensitisation occurs, discontinue use.

STORAGE

Store materials in a dry dark place. Always replace all caps immediately after use.

PACKAGING

Powder **30 g**
Liquid **20 ml**
Mixing accessories

REF: TSRL



Consult instructions for use



Flammable liquid



Temperature limit



Use by



Keep away from sunlight



Catalogue number



Keep dry



Batch code



Do not reuse



Manufacturer

Failure to comply with the conditions of storage leads to a change of the working characteristics of the material and decrease the shelf life of material. The manufacturer is not responsible for any loss of quality caused by the failure to comply with terms of transportation, storage and use established by the manufacturer for this product. Responsibility for the use of the material for purposes other than those specified by the manufacturer, the user's responsibility.

