



INSTRUCTION FOR USE

GDT GINGIVAL DAM

Light-cured Rubber Dam in Syringe

PURPOSE

Gingival Dam is viscous, light-cured, passively sealing resin in the syringe. Used to isolate and protect soft tissues. Easy to apply and remove, yet strong enough to maintain a barrier. The dark blue colour of **Gingival Dam** contrasts perfectly with gingiva and teeth, making it an ideal solution for in-office whitening.

INSTRUCTIONS FOR USE

Isolation:

1. Remove the cap and attach the tip to the syringe. Place the plastic cheek retractors.
2. Rinse and dry teeth and upper gingiva.
3. When isolating the mandibular jaw arch, use saliva ejector and cotton rolls to prevent saliva from flowing through anterior embrasures.
4. Apply the **Gingival Dam** through the embrasures onto cingulum. Light cure for 20 sec.
5. Slowly apply the **Gingival Dam** onto the gingiva to build a 5 mm wide and 2 mm thick strip. Lap 0.5 mm onto the enamel. Extend the strip beyond the last tooth to be bleached. Fill the open embrasures completely, where exist. Resin "buttons" extending through the gingival embrasures improve the retention of the barrier.
6. Light-cure for 20 sec using a scanning motion (where less than a full arch needs to be protected, scan the barrier for 3-4 sec per tooth). Check the resin for a full cure using a probe.

Clean Up:

1. After cleaning the whitening gel residues, rinse teeth with air/water syringe and vacuum.
2. Using the explorer or another suitable instrument, lift the **Gingival Dam** from the gingival surface. The material usually lifts in one piece.

- Remove interproximal leftovers using explorer and floss. Rinse with firm air/ water spray.

PRECAUTIONS AND WARNINGS

- Contains methacrylate resins. Do not use on patients with known sensitivity to resins, especially methacrylates. If allergic reaction, dermatitis or rash develops, consult a physician.
- Confirm the patient's oral health before using the product.
- The tips are disposable and intended for single use.
- To avoid cross-contamination, do not re-use tips.

STORAGE

Store in a cool, dry place (5-25°C / 41-77°F). It is important that the cap is replaced tightly after each use. Store out of reach of children.

PACKAGING

Gingival Dam in Syringe 2g	5 pcs
Tips	10 pcs
Instruction for use	1 pcs

REF: GDAM



Consult instructions for use



Use by



Caution, consult accompanying documents

REF

Catalogue number



Temperature limit

LOT

Batch code



Keep away from sunlight



Manufacturer



Keep dry

Failure to comply with the conditions of storage leads to a change of the working characteristics of the material and decrease the shelf life of material. The manufacturer is not responsible for any loss of quality caused by the failure to comply with terms of transportation, storage and use established by the manufacturer for this product. Responsibility for the use of the material for purposes other than those specified by the manufacturer, the user's responsibility.