



## INSTRUCTION FOR USE

# GDT TEMP GLUE

### Temporary Dental Glue Cement for Crowns & Bridges

#### **PURPOSE**

A ready to use temporary cement which re-fixes dental crowns, inlays and bridges. It is not a glue – it is a cement which sets only if it comes into contact with water or saliva and takes approximately 15-20 minutes to set. This cement has been developed to provide an ideal temporary solution. It does not irritate the gums, it is easy to work with and can be easily removed by a dentist without any damage to the bridge or crown. Please note that this is only a temporary solution and you must consult your dentist as soon as possible. The composition has been tested by clinicians. To ensure best results, please read the instructions carefully.

#### **INSTRUCTIONS FOR USE**

1. Using a pin or similar object remove as much as possible of the old, loose cement from inside the bridge or crown. Do not use undue force that may damage the crown. Clean the inside of the crown with warm water and soap or detergent and rinse it under running water.
2. Remove excess water with a cotton wool bud or tissue.
3. Clean the area in the mouth (tooth stump) where the crown is to be replaced. Rinse the mouth with water (do not dry the stump).
4. Apply a small amount of the cement around the inner edge of the crown or cap. Do not fill the crown. For bridges, apply a small amount of the cement around the inner edge of each crown of the bridge.
5. Press the crown in place for 3 minutes. For bridges fit the bridge back in your mouth and firmly press down with your thumb and fingers. Then carefully bite into place. You will know when the crown or bridge is correctly re-seated as you can close your teeth back together again comfortably. (If you are not satisfied with the fit, you have up to 5 minutes to remove the bridge or crown and repeat the whole process).

6. Remove any excess cement from around the base of the crown. The setting of the cement takes approximately 15-20 minutes in contact with oral moisture. Do not chew for the next 3 hours.

### **IMPORTANT**

Please remember that the cement takes approximately 15-20 minutes to set in the presence of oral moisture – it's not a regular glue. Always use small quantities and apply to the inner edge of the crown(s). This product is a temporary solution – please consult your dentist as soon as possible.

### **STORAGE**

Store in a cool, dry place (5-25°C / 41-77°F). It is important that the cap is replaced tightly after each use. Store out of reach of children.

### **PACKAGING**

Temp Glue in Syringe	7 g
Tips	5 pcs
Instruction for use	1 pcs

**REF: TMPG**



*Consult instructions  
for use*



*Use by*



*Caution, consult  
accompanying  
documents*



*Catalogue number*



*Temperature limit*



*Batch code*



*Keep away from sunlight*



*Manufacturer*



*Keep dry*

*Failure to comply with the conditions of storage leads to a change of the working characteristics of the material and decrease the shelf life of material. The manufacturer is not responsible for any loss of quality caused by the failure to comply with terms of transportation, storage and use established by the manufacturer for this product. Responsibility for the use of the material for purposes other than those specified by the manufacturer, the user's responsibility.*