



Made for GDT Implants
Yitzhak Ben Zvi 9/64
Beer Sheva, Israel
www.gdentalt.com
1.800.390.1105

INSTRUCTION FOR USE

GDT HEMOSPONGE

Absorbable sterile collagen haemostatic sponge

PURPOSE

Hemostatic antiseptic **GDT HemoSponge** is non-toxic, non-allergenic, non-immunogenic, non-pyrogenic and overall biocompatible material used for the treatment and prevention of inflammatory complications in surgical dentistry and periodontics. After tooth extraction, in particular the treatment of alveolitis and periodontal abscesses, as well as the filling of periodontal pockets after a local anti-inflammatory therapy or curettage.

FEATURES

GDT HemoSponge absorbs approximately 40-50 times of its weight and adheres easily to the bleeding site. Uniform porosity of the material guarantees a favorable haemostatic by activating the thrombocytes at the moment the blood comes in contact with the sponge matrix.

GDT HemoSponge does not have a local irritating and bio-toxic action, stimulates the regeneration of tissues in the healing stage. It does not require the doctor's intervention to remove it, full absorption happens within 3 weeks.

DIRECTION FOR USE

By removal of teeth

After the removal of the tooth or the root of a tooth it is necessary to hole sanitation, with a view to the removal of granulation or infected tissue, and bone fragments. Take a pair of tweezers one cube of sponge **GDT HemoSponge** and place it in the hole. On the top of the sponge lay a dry tampon. In the case of traumatic removal of the tooth, the top of the sponge can be sutured.

In periodontics

In the area of periodontal defects, **GDT HemoSponge** promotes the formation of blood clot formation and rapid healing of wounds.

After mechanical process of the periodontal pockets sponge or its part placed in the prepared cavity, which helps to eliminate inflammation and prevents penetration of infection in bone.

Only for local use!

PACKAGING

Sponge cube in sterile blister
Instruction for use

32 pcs
1 pcs

STORAGE

Packed in gamma irradiation sterile blisters with guaranteed sterility. Once the package is opened, contents are subject to contamination – to be used as soon as the package is opened and unused contents discarded. Do not reuse/do not re-sterilize. Store in a dry room at a temperature not more than 30°C.

Avoid contact with moisture!



Caution, consult accompanying documents.



Do not re-use/ for single use only.



Consult use instructions.



Do not re-sterilize.



Upper limit of temperature.



Do not use if package is damaged.



Keep dry.



Date of manufacture.



Medical Device.



Use by date – Expiry Date: Month & Year.



Sterilised using irradiation.



Batch code.

Rev.No. 00-03/2020