



Made for GDT Implants
Yitzhak Ben Zvi 9/64
Beer Sheva, Israel
www.gdentalt.com
1.800.390.1105

INSTRUCTION FOR USE

GDT HEMOSPONGE

Hemostatic collagen sponge with iodoform

PURPOSE

Hemostatic antiseptic collagen sponge **GDT HemoSponge with Iodoform** is used for the treatment and prevention of inflammatory complications in surgical dentistry and periodontics: after tooth extraction, in particular the treatment of alveolitis and periodontal abscesses, as well as the filling of periodontal pockets after a local anti-inflammatory therapy or curettage.

FEATURES

Sponge **GDT HemoSponge with Iodoform** consists of lyophilized collagen, in which introduced hemostatic components affecting the separate stages of blood coagulation, to stop capillary bleeding, and anesthetic components (lidocaine). As an antiseptic sponge **GDT HemoSponge with Iodoform** contains iodoform, that in contact with living tissues allocates iodine, providing antimicrobial action, enabling the formation of granulation tissue, and also has astringent and anti-inflammatory properties.

GDT HemoSponge with Iodoform does not have a local irritating and bio-toxic action, stimulates the regeneration of tissues in the healing stage. It does not require the doctor's intervention to remove it, full absorption happens after a few days.

DIRECTION FOR USE

By removal of teeth

After the removal of the tooth or the root of a tooth it is necessary to hole sanitation, with a view to the removal of granulation or infected tissue, and bone fragments. Take a pair of tweezers one cube of sponge **GDT HemoSponge with Iodoform** and place it in the hole. On the top of the sponge lay a dry tampon. In the case of traumatic removal of the tooth, the top of the sponge can be sutured.

In periodontics

In the area of periodontal defects, **GDT HemoSponge with Iodoform** promotes the formation of blood clot formation and rapid healing of wounds. After mechanical process of the periodontal pockets sponge or its part placed in the prepared cavity, which helps to eliminate inflammation and prevents penetration of infection in bone.

Only for local use!

Not for use on patients with a sensitivity to iodine drugs!

PACKAGING

Sponge cube in sterile blister
Instruction for use

50 pcs
1 pcs

STORAGE

4 years shelf-life guarantee.
Store in a dry dark cool place (+5°C...+25°C).

Avoid contact with moisture!



*Caution, consult
accompanying
documents.*



*Do not re-use/ for single
use only.*



Consult use instructions.



Do not re-sterilize.



Temperature limit.



Date of manufacture.



Keep dry.



*Use by date – Expiry Date:
Month & Year.*



Keep away from sunlight.



Batch code.